




SGX 5150-MD QUICK START GUIDE

Medical Device Gateway

WHAT'S IN THE BOX



SGX 5150 MD

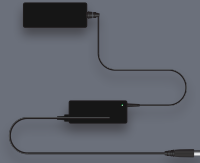

Medical Power Supply




2 External Antennas



USB Cable Type A to C

Product Views

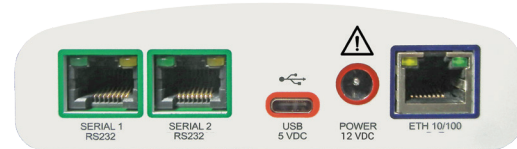

*Model shown may not reflect actual product purchased.

Thank You for Choosing Lantronix®



SAFETY WARNINGS:

- DO NOT USE IN LIFE SUPPORT OR LIFE SUSTAINING APPLICATIONS
- SAFETY OF THE SGX 5150 MD MAY BE COMPROMISED IF USED WITH POWER SUPPLY NOT PROVIDED BY LANTRONIX
- DISCONTINUE USE AND CONTACT THE MANUFACTURER IF THE PRODUCT IS DAMAGED OR EXHIBITS ABNORMAL PERFORMANCE
- POWER CONNECTION: FOLLOW THE INSTRUCTION SHEET SUPPLIED WITH THE MEDICAL POWER SUPPLY ADAPTER FOR SGX 5150 MD
- DATA CONNECTIONS: USE ONLY CABLES (USB, ETHERNET, SERIAL) AS DESCRIBED IN USER MANUAL

Wireless QuickConnect

1. Attach the two antennas and insert the medical power supply to power up the gateway.
2. From your Wi-Fi device connect to SSID SGX5150_*, where * is the gateway 12-digit serial number.
3. From your browser, connect to 192.168.0.1 and use credentials **USER ID = admin** and **PASSWORD = PASS** to login.
For security purposes, please change the admin password during initial setup.
4. Select **Wireless QuickConnect**, choose the appropriate network name for the gateway to connect with, and follow the prompts for your wireless network required security parameters.
5. Click **Apply** to save and complete the wireless network setup.

Registration

To register your gateway scan the QR code, or please visit www.lantronix.com/product-registration.

Technical Support

For the latest documentation, software, tutorials and all technical support, please visit <http://www.lantronix.com/support>.

© 2018 Lantronix, Inc. Lantronix is a registered trademark of Lantronix, Inc. All other trademarks are the property of their respective owners. Specifications are subject to change without notice. All rights reserved. 900-0812-00 Rev. A



LANTRONIX®